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### IFU URGO BOUTON DE FIEVRE - CEE

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##### • Indikations

URGO Cold sore is indicated for the protection and treatment of injuries and symptoms of labial herpes blisters and should be used from the 1st symptoms (tingling, itching, burning sensations) until the cold sore disappears completely.

URGO Cold sore is a filmogel: it forms a treating film, discreet and insulating, efficient at all stages of the cold sore, which:

- Relieves tingling, itching, burning sensations
- Limits outbreak of the cold sore
- Limits the formation of vesicles and crusts
- Promotes healing
- Prevents contamination

##### • Instructions

For people under 16 years, it must be applied by an adult. Cover the cold sore with a thin layer of product using the disposable applicator. You may feel a temporary tingling or burning sensation after application. Let dry for about 1-2 minutes to allow the formation of the film. Drying time depends on the quantity of product applied. A transparent film is formed on your cold sore. Renew the application 2 to 4 times per day maximum, when the film covers no longer your cold sore. Continue treatment until complete healing. This filmogel can be used from the first symptoms (tingling, itching, burning sensation) to limit the outbreak of the cold sore. URGO Cold sore (Labial Herpes) has been designed for a practical, safe and hygienic application.

##### • Contra-indications

Do not apply in the following cases:

- On labial herpes blisters with a diameter bigger than 1 cm which need to seek medical advice.
  - In case of bacterial infection and/or risk of bacterial infection (impetigo)
  - In case of immunodepression (due to chronic illness and/or due to chemotherapy)
  - In case of atopic eczema during herpes outbreak
  - Do not use if you are allergic to any of the ingredients.
- Because of the absence of studies in pregnant women or nursing, the use of the product is not recommended.

##### • Precautions

Application on skin only. May sting transiently during application. Do not use in children under 6 years. Do not apply near eyes. Avoid any risk of contact with eyes. Irritating to eyes. Close tightly the bottle after use. After opening, it is advisable to use the product within 6 months. Do not reuse the applicator (single use). Do not swallow. Do not smoke during application, before complete drying of the product or near the open bottle. Flammable at liquid state. Keep out of reach of children. It is not necessary to remove the residual film before a new application. If necessary, the film can be removed by reapplying product and wiping immediately with a compress. Seek doctor's or pharmacist's advice in case of doubt or:

- if you have more than 6 herpes outbreaks in a year,
- if you notice no improvement after 7 days.

##### • Storage

Do not use after the expiry date printed on the packaging.

Keep away from open flame or spark, at a temperature lower than 40 °C.

##### • Composition

Cellulose derivative, mineral acid, carboxylic acid, vegetable oil, alcohol and water.

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