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### IFU URGO APHTES TUTTI FRUTTI 6ML - CEE

Aurélia REVILLE le 24/08/2020 à 17:49

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### **GB** URGO Mouth Ulcers And small mouth wounds

Liquid dressing ready to use for the treatment of mouth ulcers (1 to 2 per crisis <1 cm in diameter) and small wounds of the oral mucosa.

#### • **What is a mouth ulcer?**

It is a superficial lesion of the oral mucosa which is manifested by a painful ulcer with sharp edges and white background, surrounded by a bright red lining. The origins are poorly understood, although some factors may promote their appearance: heredity, some foods, physical trauma (biting of the buccal mucosa, dentures), some medications, stress or hormones.

#### • **How does URGO Mouth ulcers, ans small mouth wounds work?**

Once applied, the protective film formed, visible, thin and flexible:

- **ISOLATES** from food and saliva until 4 hours
- **RELIEVES** pain quickly and durably
- **PROMOTES THE HEALING**

#### • **Directions for use**

Wipe one side of the spatula on the edge of the bottle. Apply the remaining amount using the tip of the spatula in order to completely cover the mouth ulcer or lesion. Let dry about ten seconds and then dose the mouth: a visible film is formed. If you think the film does not properly cover the mouth ulcer or lesion, renew the operation. Apply 3 to 4 times daily, without removing the film from the previous application, preferably before the main meals, until complete healing (3 to 5 days). The product's hygiene is guaranteed by its composition, it is not necessary to clean the spatula after use.

#### • **Composition**

Cellulose derivative, mineral acid, carboxylic acids, alcohol, water, flavour, sucralose.

#### • **Contra-indications**

Do not apply in case of major aphthoses (diameter> 1cm) or herpeticiformis (very small ulcers present in large numbers) that require the advice of a doctor. Do not use on infected wounds, burns, deep wounds (bleeding). Do not use if allergic to salicylate derivatives.

Use of the product is contraindicated for pregnant and breastfeeding women and children.

Product can be used only from 18 years old.

#### • **Precautions**

Always close the bottle after use. After opening, it is advisable to use the product within 6 months. To be applied exclusively on the oral mucous membrane. May cause a temporary discomfort sensation and some tingling. Do not apply on the external surface of the lips. Do not scratch off the film once formed, it will disappear by itself. If no improvement is noticed after 5 days treatment, stop the treatment and seek medical advice. Keep out of reach of children. Do not drink the content of the bottle. Do not smoke during application, before complete drying of the product or near the open bottle. In case of doubt, seek pharmacist advice. In case of mouth wounds generated by wearing a dental appliance, it is possible to use the product if the product is not covered by the dental appliance.

#### • **Storage**

Do not use after expiry date indicated on the packaging. Keep away from flame or sparks. Store at a temperature lower than 30 °C.

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