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IFU URGO CREVASSE 3,25ML- CEE

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EN URGO Cracks

Liquid dressing ready to use for the treatment of cracks, fissures and crevices on hands and feet.

• What is a crack?

It is a more or less profound disruption of the surface of the skin due to dryness or thickening of skin. The origins are mainly exposure to heat, cold, aggressive products type detergents or rubbing.

• How does URGO CRACKS act?

Once applied, the film formed, discreet and flexible creates an environment conducive to healing:

- PROTECTS from external aggressions
- BRIDGES AND AVOIDS THE REOPENING of the crack
- RELIEVES pain
- PROMOTES THE HEALING
- WATER RESISTANT

• Directions for use

For people under 16 years old, it must be applied by an adult. Apply the product using the spatula to fill and cover the crack along its entire length. Let dry for a few seconds. To be repeated 2 to 3 times daily until complete healing. Close tightly after use. If necessary, the product may be removed by re applying a drop on the film already formed, and immediately wipe with a dry pad. The hygiene of the product is guaranteed by its composition, it is not necessary to clean the spatula after use. After opening, it is advisable to use this product within 6 months.

• Composition

Triticum vulgare - Nitrocellulose - Isopropyl alcohol - Ethyl acetate - alcohol - Hydroxypropylcellulose - Ricinus communis.

• Contra-indications

Do not use on infected wounds, fungal infections, burns, and bleeding wounds. Do not use on mucous membranes (lips in particular). Because of the absence of studies in pregnant women or nursing, the use of the product is not recommended.

• Precautions

Skin application only. The product should be applied by an adult. For use on located wounds. Keep out of reach of children. Do not use in children under 3 years old. Do not smoke during application, before complete drying of the product or near the open bottle. Can sting at application. If in doubt consult your pharmacist. Do not swallow. Avoid contact with eyes. Irritating to eyes. The product can be used by diabetic subject to prior consultation of a doctor.

• Storage

Do not exceed the expiry date indicated on the packaging. Keep away from flame or sparks. Store at a temperature lower than 25 °C.

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