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### IFU URGO ONGLES ABIMES 3,3 ML - CEE

Marine THIBAUDIN VERCRUYSSSE le 03/09/2020 à 14:36

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### **URGO Filmogel® Damaged Nails By Mycosis and mild Traumatism**

**• What are nails made of and how do they grow?** The nails are keratin formation, of a thin layer (0.5-1.3 mm) of dead skin cells. These cells are continuously being produced by the nail matrix at the base of the nail. These cells constitute unguis tableti. Nail's growth differs from one individual to another, but healthy nails grow at an average of 0.5 to 1.2 mm per week.

**• The damaged nails:** Damaged nails are a very common problem and can have different causes. There may be physical causes (over washing hands or wearing tight shoes), or chemical causes (use of detergents and solvents) which can allow fungal infection (mycosis). Damaged nails can be white or yellow and have sharp edges.If you want to know the specific reason, it is advisable to consult your dermatologist or doctor.

**• URGO Damaged nails Filmogel® was created to keep your nail nice and healthy.** URGO Damaged Nails Filmogel®, is indicated for the protection and treatment of nails damaged by mycosis and mild traumas, which are defined as damages which do not lead to the removal of the nail and without interruption of integrity of nail's tableti. URGO Damaged Nails Filmogel® contains hydroxypropylchitosan. The hydroxypropylchitosan is a polymer of natural origin (from shells of crabs) that will interact with the structure of the nail, with a reinforcing action. Once applied, URGO Damaged Nails Filmogel® will form a protective and stabilizing film on the surface of the nail, which will protect it from microbiological, chemical and physical attacks. The product will promote the regrowth of your healthy, regenerated and protected nails.

**• Instruction for use:** - Apply the product with the brush on clean and dry nails on the surface of the nail. - Apply the product once a day, preferably before bedtime. The film is water soluble, so we advise you to apply it at bedtime, allowing the product to remain on the nail for an optimal period of time. - The product does not require filing or use of solvent to be removed.

**• Follow up of the treatment:** The results of treatment by URGO Damaged Nails will be noticed with the regrowth of the nail; do not expect an immediate visible result. The sooner the damage is treated, the faster it will disappear! The nail grows at an average 1 cm per quarter for the hands and 1/3 cm for the feet. With age, the regrowth is slower.

**• Warnings and precautions:** We recommend to consult a doctor if: - The lower part of your nail looks affected (risk of aggravation of the trauma on the nail) (dotted area on the drawing). - Your nail is black without having suffered from any trauma. - You have diabetes or immunodeficiency (risk of infection).

Do not swallow, to be apply exclusively on nail. Keep away from flame or sparks (risk of burns). Do not smoke during application, before complete drying of the product or near the open bottle (risk of burns). Keep out of reach and sight of children. The product is not a nail varnish. You can, if desired, apply over the protective film coating a cosmetic varnish. Do not forget to remove the nail varnish before re-applying the product.

**• To reduce any risks of infection or cutaneous reactions:** For local application, on the nail. Strictly observe the instructions for use. Do not exceed the expiration date indicated on the packaging. Do not use a damaged product (deteriorated brush, solution made solid). Close tightly after use.

**• Contra-indications:** Do not use on wounds (risk of pain or infection). - Do not use on psoriasis or eczema - Do not use on brittle or cracked nails. - Do not use in case of per-ungual skin damage - Do not use on nails with black or brown pigmentations, located on or near the nails. - Do not use if you are allergic to any of the ingredients of Urgo Damaged Nails. - Do not use in children under 3 years old.

**• Side effects:** In very rare cases, skin irritation can occur around the area of application.

Any serious adverse event related to this product must be declared to Laboratoires URGO HEALTHCARE, distributor and national health authority in your country.

**• Components:** hydroxypropylchitosan (animal origin), purified water, ethanol, preservative agent: propylene glycol, Octopinox®.

Medical device.

Date of issue CE marking : 2020 - Latest revision of the leaflet V3: 08-2020

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